

CARE AND DISINFECTION OF THE CHAMBER AND GURNEY

- Clean the chamber according to the type of cases being treated and as directed by the medical staff. Refer to the list of acceptable cleaners available from SECHRIST INDUSTRIES.
- Wash Stretcher, Pillows, Positioning Pads and Mattress surfaces with approved cleaner or mild dish washing soap. Refer to the Gurney User Manual for cleaning instructions.

CARE AND DISINFECTION OF THE CHAMBER AND GURNEY

- Use the same methods for cleaning as used for the Acrylic Cylinder.

APPROVED CHAMBER CLEANERS

DISINFECTANT	MANUFACTURER CITY, STATE	MANUFACTURER PART NUMBER	RECOMMENDED DILUTION WITH WATER	STRENGTH APPROVED FOR USE	APPROVED AT RECOMMENDED DILUTION
Asepti-HB	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	Not Provided	Ready to use	N/A
Virasept	Ecolab Professional Products Ecolab Inc. St. Paul, MN	6002314	Not Provided	Ready to use	N/A
Oxycide Daily Disinfectant	Ecolab Professional Products Ecolab Inc. St. Paul, MN	6000189	1:128-1:64	1:128-1:64	YES
Neutral Disinfectant Cleaner	Ecolab Professional Products Ecolab Inc. St. Paul, MN	6027314	1:256-2:64	1:256-2:64	YES
Tor-HB (Discontinued, can use <u>unexpired</u> stock)	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:64	1:64	YES
Stat III TB	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:128	1:128	YES
Beaucoup	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:128	1:128	YES
LpH-se	Calgon, Vestal Division STERIS Corp. Mentor, OH	N/A	1:256	1:256	YES
Sani-Cloth Bleach Wipe	PDI/Nicepak Orangeburg, NY	U13295	Equivalent to 1:10 dilution	Ready to use	N/A
Clorox Healthcare Bleach Germicidal Cleaner	Clorox Professional Products (Oakland, CA)	68967, 68970, 68832, 68973, 68978	Not provided	Ready to use	N/A
Dispatch with Bleach	Caltech Industries, Inc. Midland, MI	68832	Not provided	Ready to use	N/A
Quaternary Disinfectant Cleaner	ECOLAB® St Paul, MN	6027204 6063304	1:256-1:64 Not provided	1:256 - 1:64 Ready to use	YES N/A
Coverage® Spray HB Plus	Steris Corporation Mentor, OH	1624-77	Not provided	Ready to use	N/A
Sani-Cloth® HB	PDI/Nicepak Orangeburg, NY	Q08472	Not Provided	Ready to use	N/A