

CARE AND DISINFECTION OF THE CHAMBER AND GURNEY

- Clean the chamber according to the type of cases being treated and as directed by the medical staff. Refer to the list of acceptable cleaners available from SECHRIST INDUSTRIES.
- Wash Stretcher, Pillows, Positioning Pads and Mattress surfaces with approved cleaner or mild dish washing soap. Refer to the Gurney User Manual for cleaning instructions.

CLEANING OF PAINTED AND ANODIZED SURFACES

- Use the same methods for cleaning as used for the Acrylic Cylinder.

APPROVED CHAMBER CLEANERS

DISINFECTANT	MANUFACTURER	EPA REGISTRATION NUMBER	RECOMMEND ED DILUTION WITH WATER	STRENGTH APPROVED FOR USE	APPROVED AT RECOMMENDED DILUTION
Asepti-HB	Ecolab Professional Products Ecolab Inc.	61178-2-1677	Ready to Use	Ready to use	N/A
Virasept	Ecolab Professional Products Ecolab Inc.	1677-226	Ready to Use	Ready to use	N/A
Oxydice Daily Disinfectant	Ecolab Professional Products Ecolab Inc.	1677-237	1:128-1:64	1:128-1:64	YES
Neutral Disinfectant Cleaner	Ecolab Professional Products Ecolab Inc.	47371-129	1:256-2:64	1:256-2:64	YES
Stat III TB	Ecolab Professional Products Ecolab Inc.	39967-89-2677	1:128	1:128	YES
LpH-se	Calgon, Vestal Division STERIS Corp.	N/A	1:256	1:256	YES
Sani-Cloth Bleach Wipe	PDI/Nicepak	9480-8	Equivalent to 1:10 dilution	Ready to use	N/A
Clorox Healthcare Bleach Germicidal Cleaner	Clorox Professional Products	67619-12	Ready to Use	Ready to use	N/A
Quaternary Disinfectant Cleaner	ECOLAB Professional Products	6836-78	1:256-1:64	1:256 - 1:64 Ready to use	YES
Quaternary Disinfectant Wipes	ECOLAB Professional Products	6836-372	Ready to Use	Ready to Use	N/A
Coverage® Spray HB PLus	Steris Corporation	6836-152	Ready to Use	Ready to use	N/A
Sani-Cloth® HB Wipe	PDI/Nicepak	61178-4-9480	Ready to Use	Ready to use	N/A
Virex II 256	Diversey	70627-24	1:128	1:128	YES
Virex Plus	Diversey	6836-349	1:128	1:128	YES
Peroxide Multi Surface Cleaner and Disinfectant RTU	ECOLAB Professional Products Ecolab Inc	1677-251	Ready to Use	Ready to Use	N/A
Vital Oxide	Vital Solutions	82972-1	Ready to Use	Ready to Use	N/A